

Hotkeys / Keyboard-Shortcuts

2017-12-04



CTRL + middle-mouse button hides the object under the pointer, then CTRL + Shift + middle-mouse button makes it visible again

Shift + middle-mouse button toggles transparency of object under the pointer

Show/Hide groups (Group Selector) hotkeys:

A: Antagonists	S: Jaw Scans	X: Upper jaw (MaXillae) restorations in both jaws only	N: Lower jaw (MaNdible) restorations in both jaws only
G: Gingiva scans	E: Anatomic parts	F: Full contour parts	R: Reduced parts
C: Connectors	W: Waxup scans	I: 2D images	D: DICOM
P: Pre-op (situ)	V: Virtual gingiva design	B: Crown bottoms	M: Merged parts
T: Telescopes	O: Other	Shift + hotkey to toggle transparency	

(Hotkeys are shown in the tooltip of the checkbox in Show/Hide Groups)

Undo/Redo buttons support **CTRL + Z / CTRL + Y**

Show hide controls:

CTRL + D: Show Distances tool	CTRL + R: Measurement tool	CTRL + P: Show View Clip Plane tool
--------------------------------------	-----------------------------------	--

Wizard:

F1: Help	CTRL + Space: Next	CTRL + Backspace: Back
-----------------	---------------------------	-------------------------------

In all design controls:

Tab: switch to next tab	Shift + Tab: switch to previous tab
--------------------------------	--

Crown/inlay/veneer bottom:

1 / 2 / 3: activate brush for zone 1,2,3

Freeform:

1 / 2: activate add/remove/smoothing	3 / 4 / 5 / 6: activate move cusps, tooth parts, entire tooth, ridge		
Ctrl + X: cut intersections with antagonist + gingiva + adjacents	Ctrl + N: cut intersections with adjacents only	Ctrl + G: cut intersections with gingiva only	Ctrl + A: cut intersections with antagonist only
Ctrl + Q: toggle "Hold eQuator"	Ctrl + T: toggle "Hold cusp Tips"	Ctrl + M: toggle movement restriction (occlusal only)	

Preparation margin detection:

1: detect/Add point	2: remove point	3: edit margin	4: move points up/down
5: (4 if "Move points" is invisible): draw free	Ctrl + X: clear	+ / -: move up/down	

Full Denture Model Analysis: **Enter** for "Next", **Backspace** for Back